



**EXPANDING THE  
BOUNDARIES OF  
HEALTHCARE**

# **Horus Scope for Vidyo CLINiC™ Gen 2 Installation and User Guide**

**Iron Bow Part Number:  
ACC-CRADLE-VIDYO-BO1**

**For use with:  
CLINiC-V-10XIR-B02**

**Document Number: DOC-UG-CRADLE-S1-V-B**

**Version 3.0**

**3/01/2021**

**Copyright © 2021 Iron Bow Technologies**

All Rights Reserved.

Specifications subject to change without notice.

For general inquiries, contact:

**Iron Bow Healthcare Solutions**

2303 Dulles Station Boulevard, Suite 400

Herndon, VA 20171

Toll: 800.338.8866

Tel: 703.279.3000

[www.ironbowhealthcare.com](http://www.ironbowhealthcare.com)

For support, contact:

**Iron Bow Client Service Center**

Toll: 833.476.6269 (833.IRONBOW)

Email: [CSC@ironbow.com](mailto:CSC@ironbow.com)

# Contents

---

<b>Introduction .....</b>	<b>5</b>
<b>Installing the Horus Scope Cradle .....</b>	<b>6</b>
Preliminary Steps .....	7
Installing Cradle in Side-Mounted Position .....	8
Moving Cradle from Side to Rear Mounting Position .....	11
<b>Operating the Horus Scope.....</b>	<b>14</b>
Horus Scope Functions and Components .....	<b>Error! Bookmark not defined.</b>
Horus Scope Functions and Components .....	14
Basic Horus Scope Operations.....	15
Changing Horus Scope Lens Attachments .....	16
Troubleshooting Tips .....	16

# Figures

---

Figure 1. CLINiC with Horus Scope and Cradle assembly.....	5
Figure 2. Horus Scope cradle attachment parts .....	6
Figure 3. Horus Scope – CLINiC Connectivity Diagram .....	7
Figure 4. Fully Assembled CLINiC 10x (rear view) .....	7
Figure 5. Horus Scope Components (front view).....	<b>Error! Bookmark not defined.</b>
Figure 6. Horus scope components (rear, side and bottom views) .....	<b>Error! Bookmark not defined.</b>
Figure 6. Horus Scope components (front view) .....	14
Figure 7. Horus scope components (rear, side and bottom views) .....	14

## Safety & Maintenance

Please read these instructions completely before operating the equipment and keep this manual for future reference. The Horus Scope cradle assembly is designed for use with Iron Bow CLINiC systems and should not be used with any other system. The information in this manual is intended for persons who install and operate the equipment as well as repair or servicing personnel,

It is recommended that the Iron Bow CLINiC is used with an AC isolation transformer which is incorporated into the optional stand available for this system, or with a mobile cart with and internal isolated AC supply. Prior to installation and start-up of Horus scope, carefully read the user manual. As with all technical devices, the proper function and safety operation of this device depends on the user complying with the safety and cleaning recommendations presented in these operating instructions. In addition, please make sure it does not appear damaged or broken. If there are breaks on the outer casing or other visual defects, please contact the manufacturer or a certified service facility. The Jed Med Horus Scope user manuals can be found at: <https://www.jedmed.com/pages/horus-scope-manuals>

# Introduction

---

This user guide covers basic Horus Scope operation instructions, as well as installation steps of a Horus Scope cradle assembly. The cradle assembly allows convenient and safe positioning of the Horus Scope on a CLINiC.

The instructions described in this guide apply to the Vidyo CLINiC Gen 2, part number CLINiC-V-10X24-B01.



**Figure 1. CLINiC with Horus Scope and Cradle assembly**

You can find additional CLINiC resources and information about support and other related telehealth services at [www.ironbowhealthcare.com](http://www.ironbowhealthcare.com).

# Installing the Horus Scope Cradle

---

The components of the Horus Scope cradle assembly are shown in the following figure.

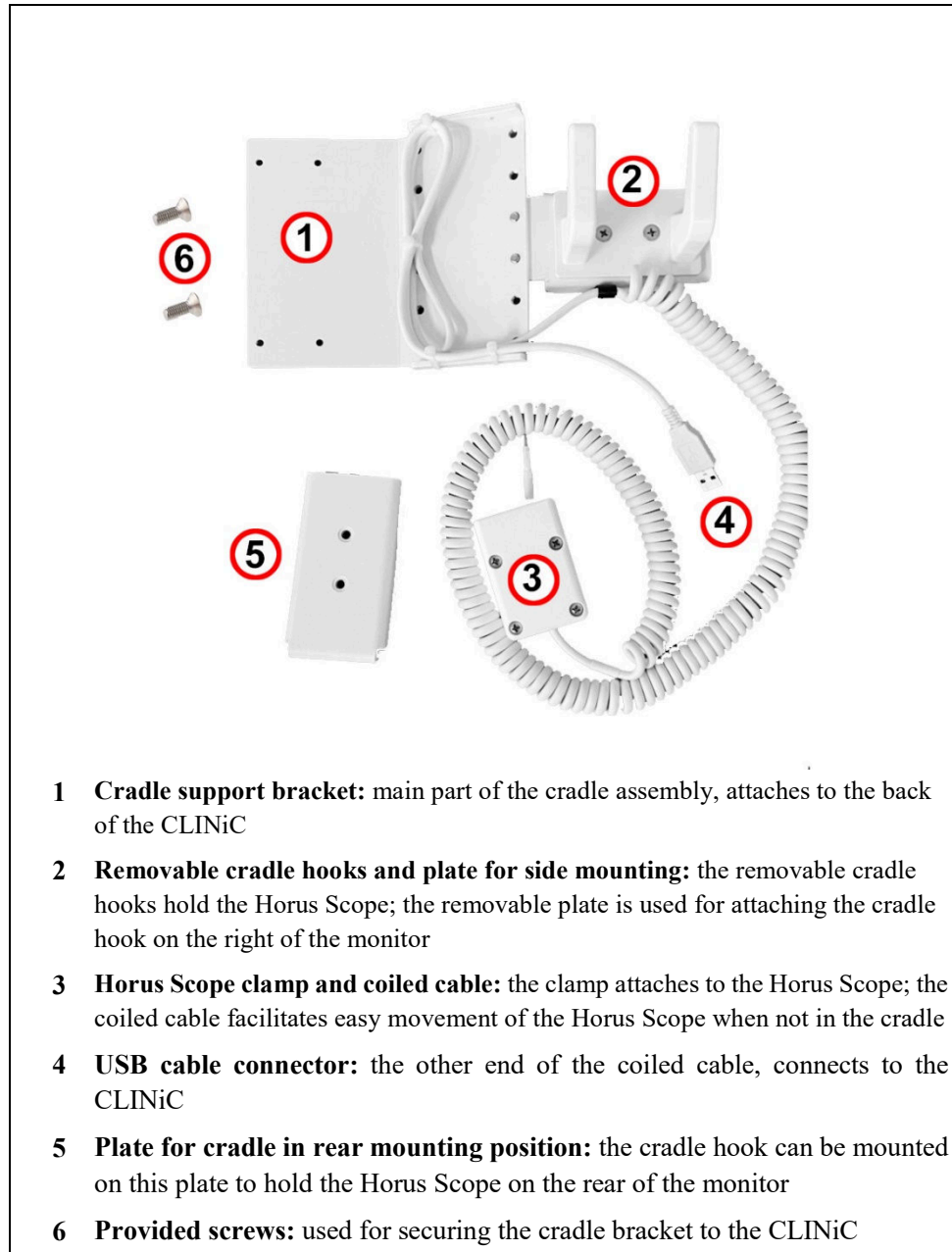


Figure 2. Horus Scope cradle attachment parts


The connection diagram between the Horus Scope and the CLINiC is shown below:



**Figure 3. Horus Scope – CLINiC Connectivity Diagram**

## Preliminary Steps

Prior to installing the Horus Scope cradle onto the CLINiC, the CLINiC should be fully assembled, as shown below (the camera should be already mounted on the CLINiC).

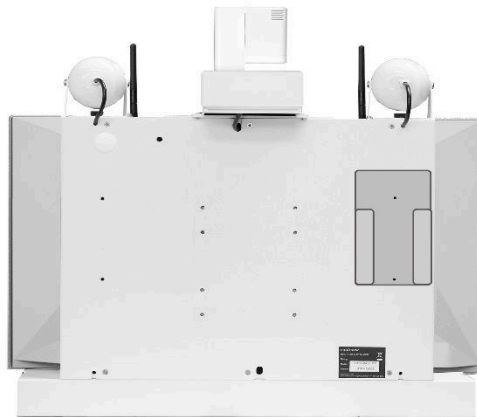


**Figure 4. Fully Assembled CLINiC 10x (rear view)**

Prior to attaching the clamp and coiled cable to the Horus Scope, make sure the Horus Scope has a working battery. To insert or replace a battery, see the instructions below.

**Note:** If the Horus Scope and the cradle were purchased together, the battery will be preinstalled in the Horus Scope. If that is the case, skip to the cradle assembly instructions.

***To install the Horus Scope battery:***

1. Open the battery cover by inserting your finger or a pointed tool in the gap at the bottom of the battery cover. Remove the battery cover by lifting it up.
2. Insert the battery with the positive end facing up.

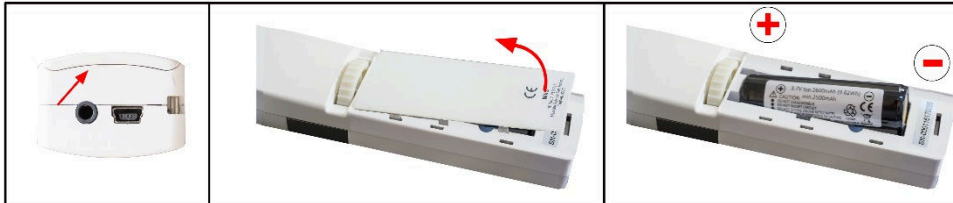


## Installing Cradle in Side-Mounted Position

The cradle can be mounted in two positions relative to the CLINiC. Side mounting; provides easy access on the right hand side of the CLINiC, which may be ideal for frequent use. Rear mounting stores the Horus Scope on the back of the CLINiC, which reduces the overall width of the system, allowing greater mobility when mounted on a cart.

**Note:** A #2 Phillips head screwdriver will be required for the installation.

***To install the cradle in Side Mounted position:***


1. Vertically align the cradle mounting bracket with the holes on the back of the CLINiC, and attach using the provided screws.

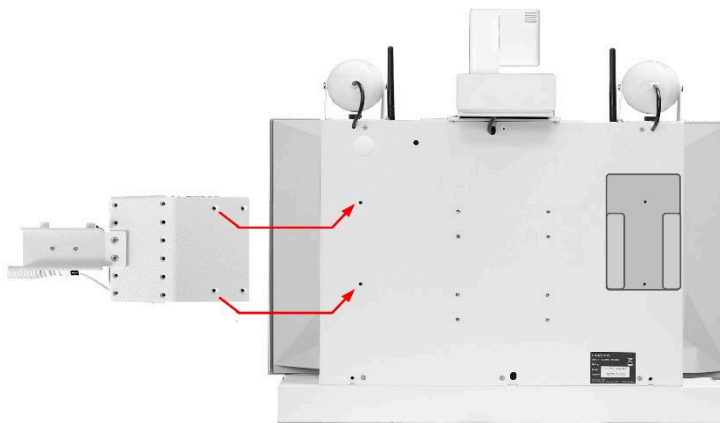


If the Horus Scope and the cradle assembly were purchased together, the Horus Scope will come pre-installed in the clamp. If this is the case, skip to step 6.



2. Remove the screws from the Horus Scope clamp (1) to expose the Mini USB connector (2).

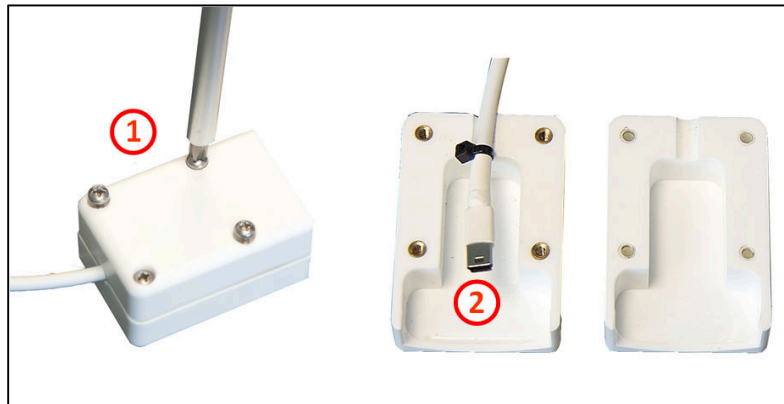


3. Place the Horus Scope in the bottom half of the clamp (the half that has metal screw holes), as shown. Insert the Mini USB connector into the corresponding port at the bottom of the Horus Scope.

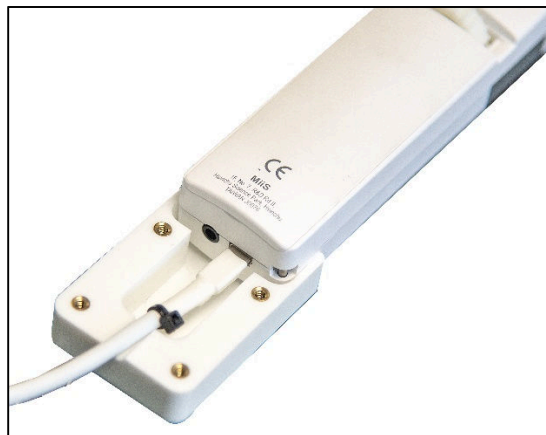


4. Secure the top side of the Horus Scope clamp and fasten the four screws.



5. Connect the USB connector end of the coiled cable to the USB port on the CLINiC, as shown.

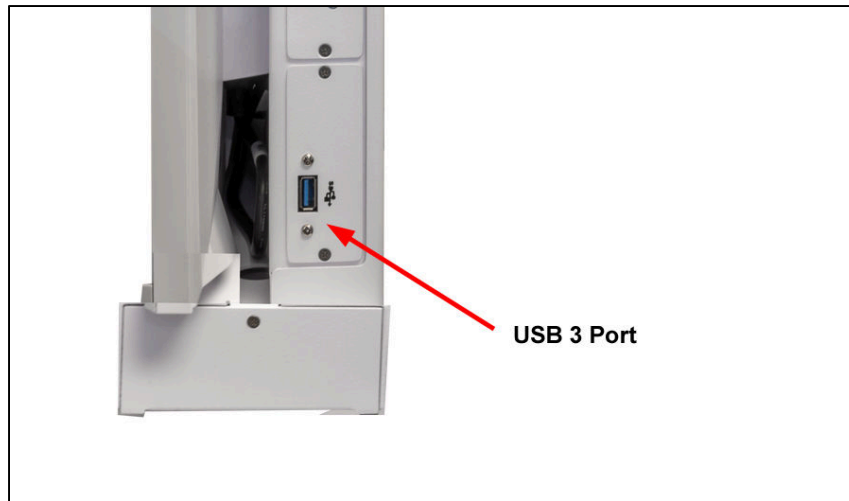

6. Place the Horus Scope in the cradle, as shown.

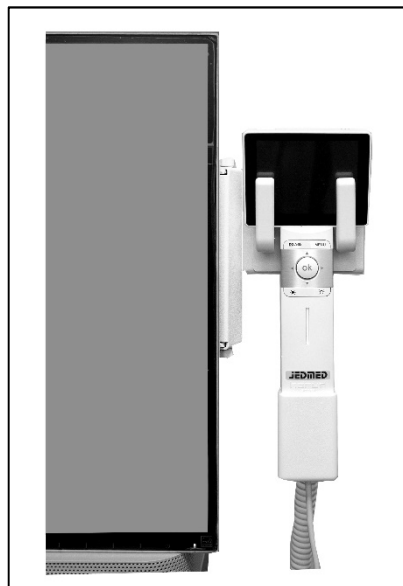


## Moving Cradle from Side to Rear Mounting Position


The instructions below describe how to change the cradle from side mounting to rear mounting. To change from rear mounting to side mounting, reverse the instructions.

**Note:** A #2 Phillips head screwdriver will be required for the installation.


1. Remove the screws attaching the cradle hooks to the position one plate. You will reuse the cradle hooks and screws later in this procedure when you will attach the hooks to the position two plate.

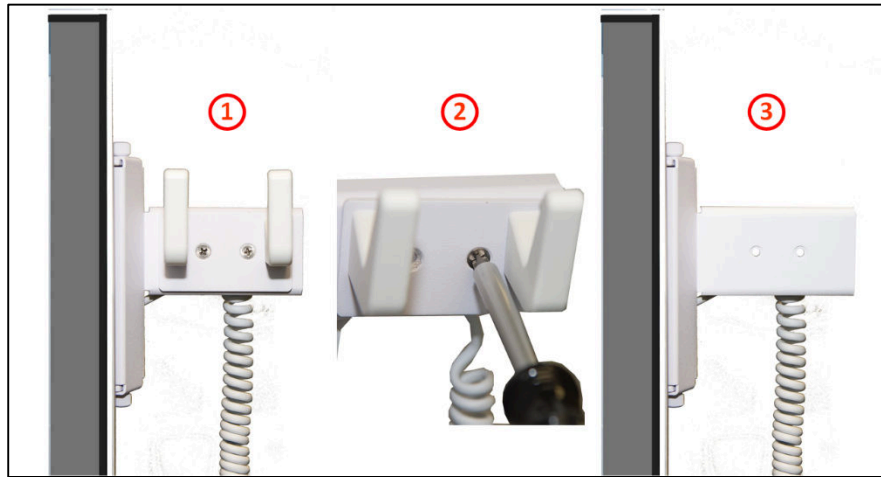


2. Remove the two screws attaching the side position plate to the cradle support bracket. You will use the same screws to attach the rear position cradle support bracket later in this procedure.

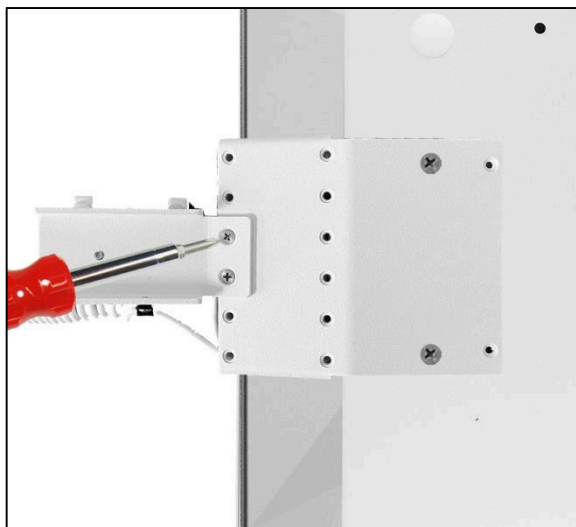

3. Remove the strain relief clamp holding the coiled cable to the side position mounting plate.
4. Fasten the strain relief clamp to hold the coiled cable to the rear position plate.

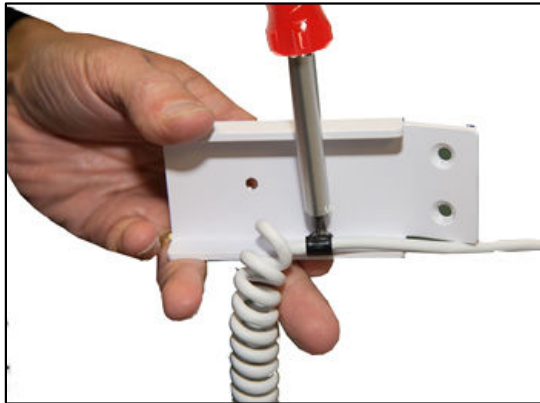

5. Place the rear position plate on the back of the cradle support bracket and secure with the same two screws that were used to secure the position one plate.

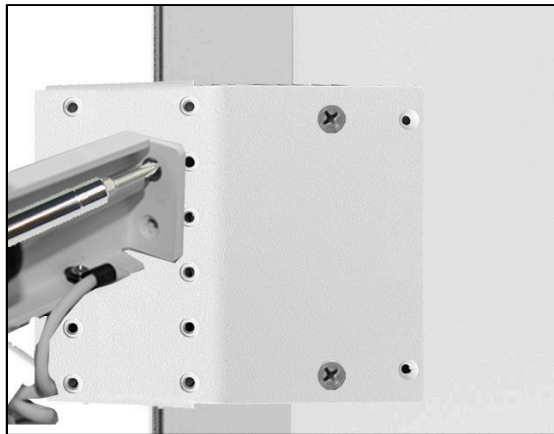

6. Fasten the cradle hooks onto the rear position plate.



7. Place the Horus Scope into the cradle on the back of the CLINiC.



# Operating the Horus Scope

The Horus Scope is used to capture and transmit images and video to a remote consulting clinician through the CLINiC. This section includes a summary of the Horus Scope functions and components, basic operation and troubleshooting tips.

## Horus Scope Functions and Components

The Horus Scope functions and device components are listed below.

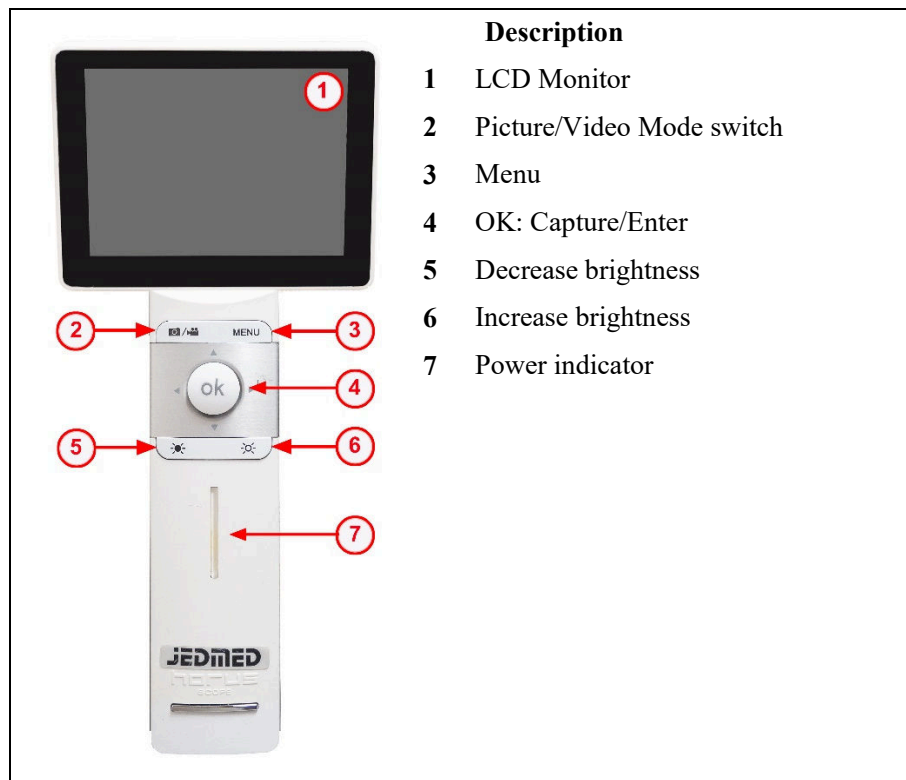


Figure 5. Horus Scope components (front view)

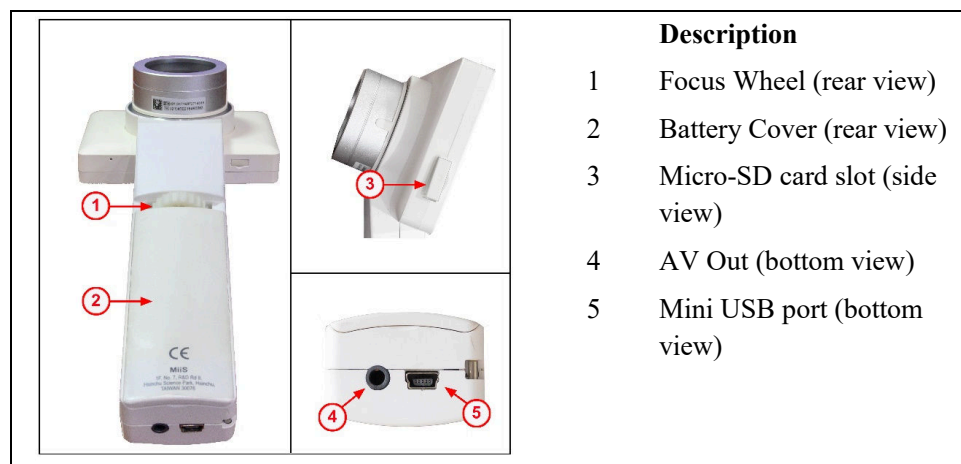


Figure 6. Horus scope components (rear, side and bottom views)

## Basic Horus Scope Operations

### *To power on the Horus Scope:*

- Press and hold the power button at the top of the unit for two seconds. The Horus Scope will start up. Start-up takes about 20 seconds.



Once the start-up procedure is complete the Horus Scope screen will display the live video feed captured by the lens currently attached to the unit.

### *To switch between live video and image capture:*

1. Press the **OK** button on the unit to capture images.



2. Press the   button to return to live view.

## Changing Horus Scope Lens Attachments

### *To remove optical lens attachment from the Horus Scope:*

1. Hold the camera in one hand and rotate optical lens counter-clockwise with the other hand until the lens detaches.
2. Gently pull to detach lens.


### *To add optical lens attachment to the Horus Scope:*

1. Hold the camera in one hand and the lens in the other hand
2. Align the ● marks on the optical lens and the camera body
3. Push the lens and the camera together
4. Rotate optical lens clockwise until secure

For more details regarding the Horus Scope features, menu options, operation and care instructions please consult the *JEDMED Horus Scope User's Manual*.

For additional information regarding the Horus Scope operation with the CLINiC, please refer to the CLINiC User Guide.

## Troubleshooting Tips

 <p>Connected Peripheral Device button on the CLINiC control panel</p>	<ul style="list-style-type: none"><li>• Press the Connected Peripheral Device button (pictured to the left) to display the Horus Scope video on the CLINiC. <b>Note:</b> The Connected Peripheral Device button becomes active and can be used after the Horus Scope is powered-up, connected to and detected by the CLINiC.</li><li>• If you see a JEDMED logo instead of the expected image, ensure that the lens is properly attached to the Horus Scope handle. Consult the User Guide included with the Horus Scope for detailed installation and operation instructions of the Horus Scope device.</li></ul>
--	--



- This Page Intentionally Left Blank -

- This Page Intentionally Left Blank -

- This Page Intentionally Left Blank -



2303 Dulles Station Boulevard, Suite 400  
Herndon, VA 20171

Toll: 800.338.8866  
Tel: 703.279.3000

[www.ironbowhealthcare.com](http://www.ironbowhealthcare.com)